# Soluciones Ascom

*Diciembre, 2025*

**ascom**

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**300**  
million  
2023 Revenue (in CHF)

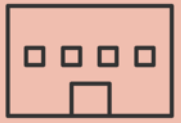
**+300**  
Channel Partners  
worldwide

**+17,000**  
sites with an Ascom SW solution

**+1.9**  
million  
mobility devices sold

**19**  
Ascom  
offices

**40**  
Nationalities

Ascom Group HQ  
**Baar, Switzerland**

**+8,500**  
hospital sites with Ascom solutions

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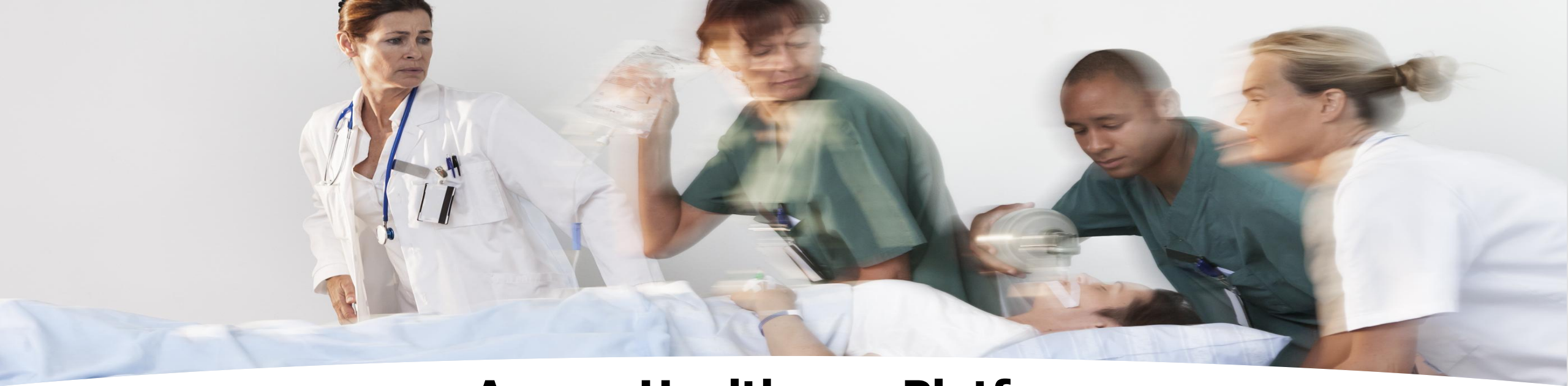
# Ascom Healthcare Platform

*Unlocking the Power of Digital Clinical Information*

## Integrate

- Connecting care devices and hospital information systems to help clinicians unlock valuable care information.

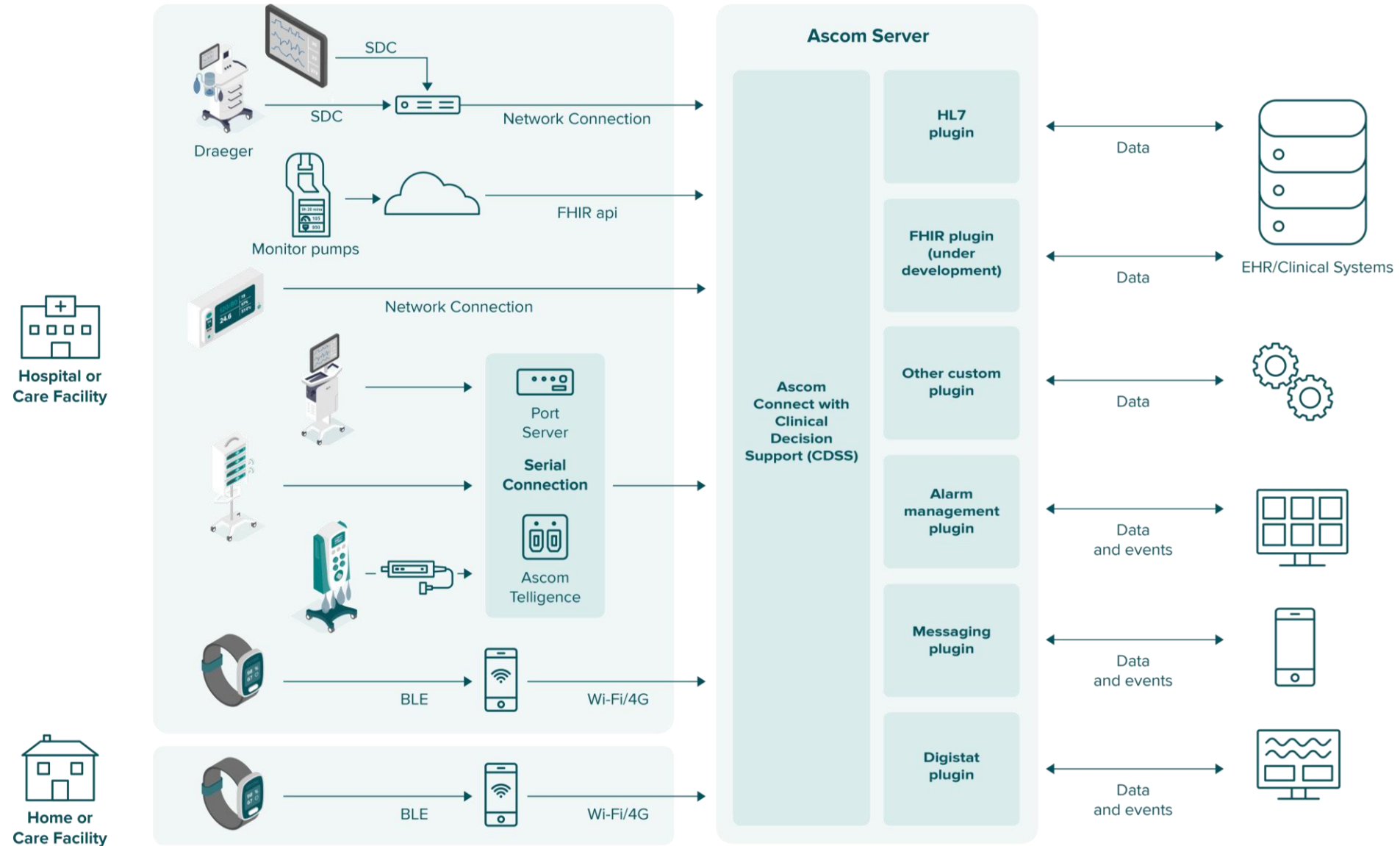
## Orchestrate

- Filtering, prioritizing and extending the flow of information to clinicians for responsive, informed decisions

## Enable

- Providing structured data to add context that enhances timely care coordination and communication across mobile teams, to address patient needs.

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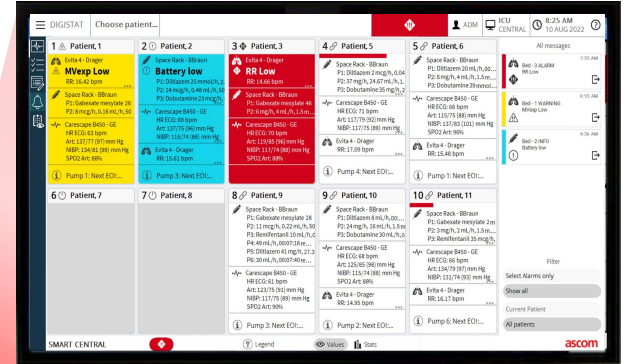
The image shows a 'Certificate of Compatibility' document. At the top center, there is a traffic light icon with three red circles. The text in the center reads: 'This is to certify that "Shenzhen Mindray Bio-Medical Electronics Co., Ltd." products have been successfully verified for Ascum Digistat Compatibility'. Below this is the 'ascom COMPATIBLE PRODUCT' logo. At the bottom left, it says 'ASCUM UMS srl - Via Amilcare Ponchielli 29 - 50018 Scandicci (FI), Italy. Digistat is a registered trademark of Ascum UMS srl'. At the bottom right, it says 'ascom' and 'ASCUM UMS srl - Via Amilcare Ponchielli 29 - 50018 Scandicci (FI), Italy. Digistat is a registered trademark of Ascum UMS srl'.

**CERTIFICATE OF COMPATIBILITY**

Ascum – Compatible Product Program ensures compatibility between the Ascum and 3rd party vendor products.

Digistat is a software as medical device (SaMD) compliant with Medical Device Directive 93/42/EEC and subsequent amendment. The following device has been tested for compatibility with the Digistat suite:

**eGateway**

Supported connectivity

Device type	Patient monitor, Anesthesia Devices
Manufacturer	Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
Model	eGateway
Connection	Ethernet
Protocol versions	6.4, 6.5

Supported in Digistat

Digistat suite	Version 5.1 and subsequent
----------------	----------------------------

Supported capabilities

Distributed Information System (DIS)*	Yes
Reliable notification of alarms (RASC/DAS)*	No
Silent ICU	No
Remote silencing of alarms	No

\* according to IEC/TR 80001-2-6:2014

**Conclusion**

Ascum UMS srl declares that the aforementioned devices conforms to the safety and connectivity requirements of the Digistat suite.

The Digistat certification is only valid for the models and versions verified - as shown in the certificate - and cannot be automatically extended to other models, protocols or versions.

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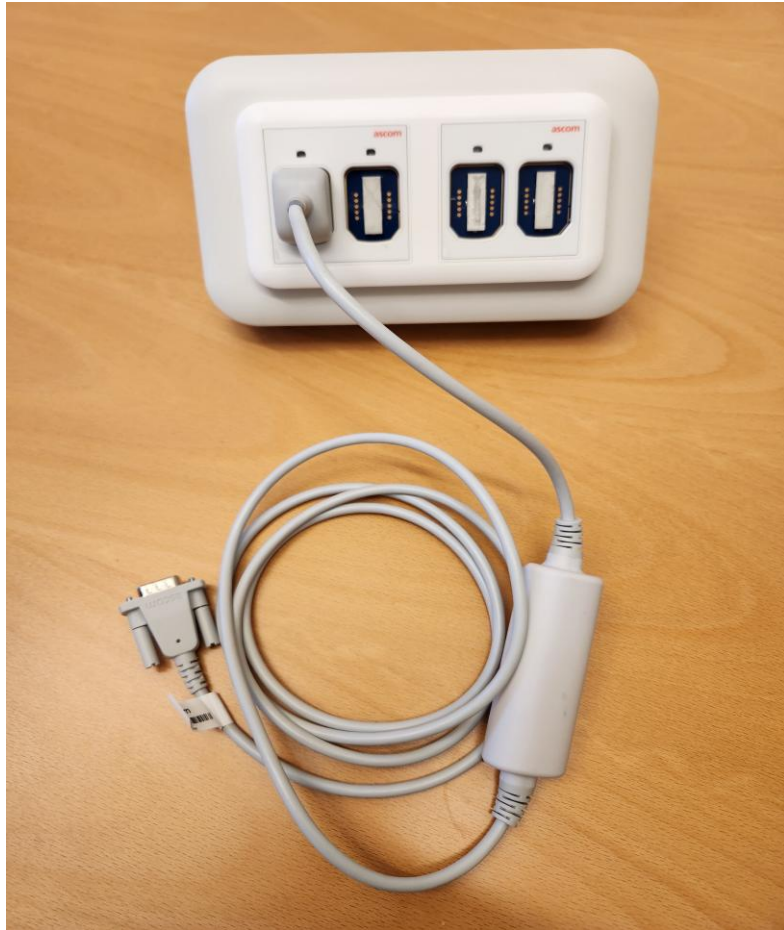
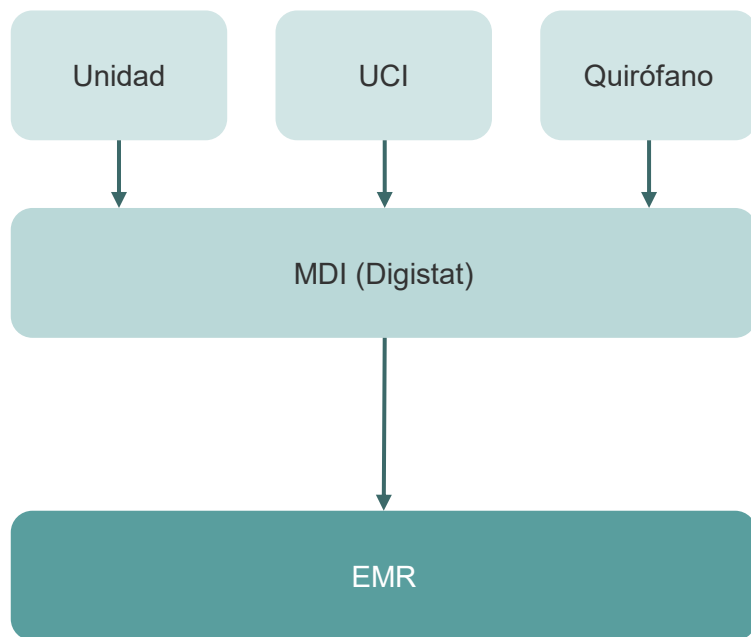
Time	Device	#	Description
4:53 PM	-		Arterial Blood Pressure too high
4:53 PM	-		ECG Sensor Warning
4:53 PM	-		Arterial Blood Pressure too high
4:53 PM	-		ECG Sensor Warning
4:53 PM	-		Arterial Blood Pressure too high
4:53 PM	-		Arterial Blood Pressure too high
4:51 PM	-		Arterial Blood Pressure too high
4:51 PM	-		ECG Sensor Warning
4:51 PM	-		ECG Sensor Warning
4:51 PM	-		Arterial Blood Pressure too high
4:46 PM	-		ECG Sensor Warning
4:46 PM	-		ECG Sensor
4:44 PM	-		ECG Sensor Warning
4:43 PM	-		ECG Sensor
4:37 PM	[syringe]	1	Bolus; Duration= 5 sec; Type= HandFree; Rate= 800 mL/h; Volume= 1.1
4:36 PM	[syringe]	1	Bolus; Duration= 5 sec; Type= HandFree; Rate= 800 mL/h; Volume= 1.1
4:36 PM	[syringe]	1	Bolus; Duration= 5 sec; Type= HandFree; Rate= 800 mL/h; Volume= 1.1

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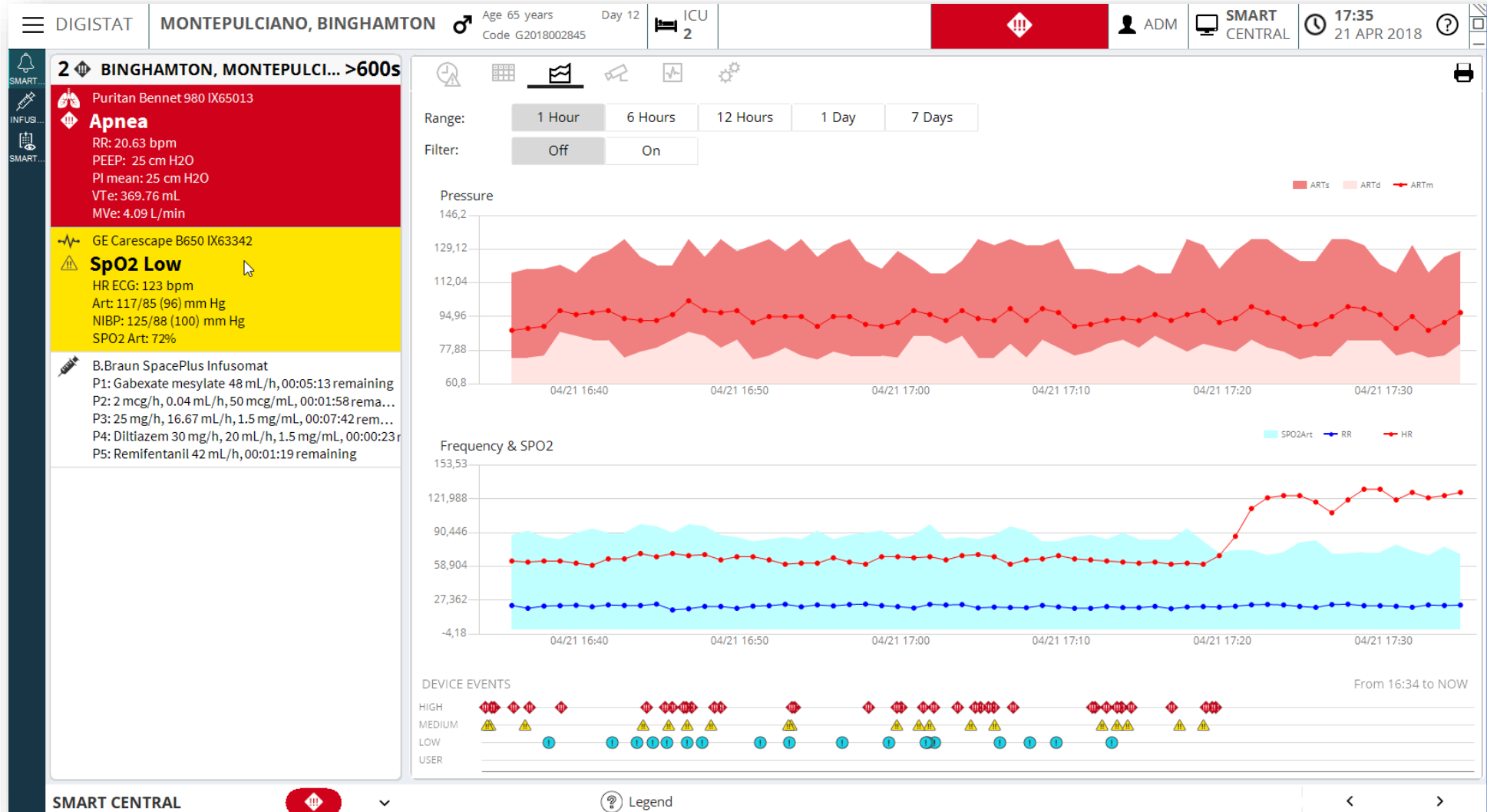
The screenshot shows the Smart Central interface with three patient cards. The middle card for 'test' displays a 'CO2 Zeroing Failed' alert. A 'DASHBOARD' window is open, showing a date range from 08/06/2022 08:25:34 to 10:25:34. To the right, an 'Event statistics' report is displayed, featuring pie charts for 'Alarm priority' and 'Source', and a table for 'Event summary'.

Device	Priority	Episodes	Episodes/ day	%	Total duration (min)	Avg. duration (min)
Total	HIGH	1105	25.0	51	315.5	0.3
	MEDIUM	820	18.0	37	4022.2	4.9
	LOW	236	5.0	10	115.5	0.5
EutaVentilator	HIGH	136	3.0	65	67.8	0.5
	MEDIUM	24	0.0	11	12.0	0.5
	LOW	47	1.0	22	23.5	0.5
Perfora Space	HIGH	1	0.0	0	0.8	0.8
	MEDIUM	0	0.0	0	0.0	0.0

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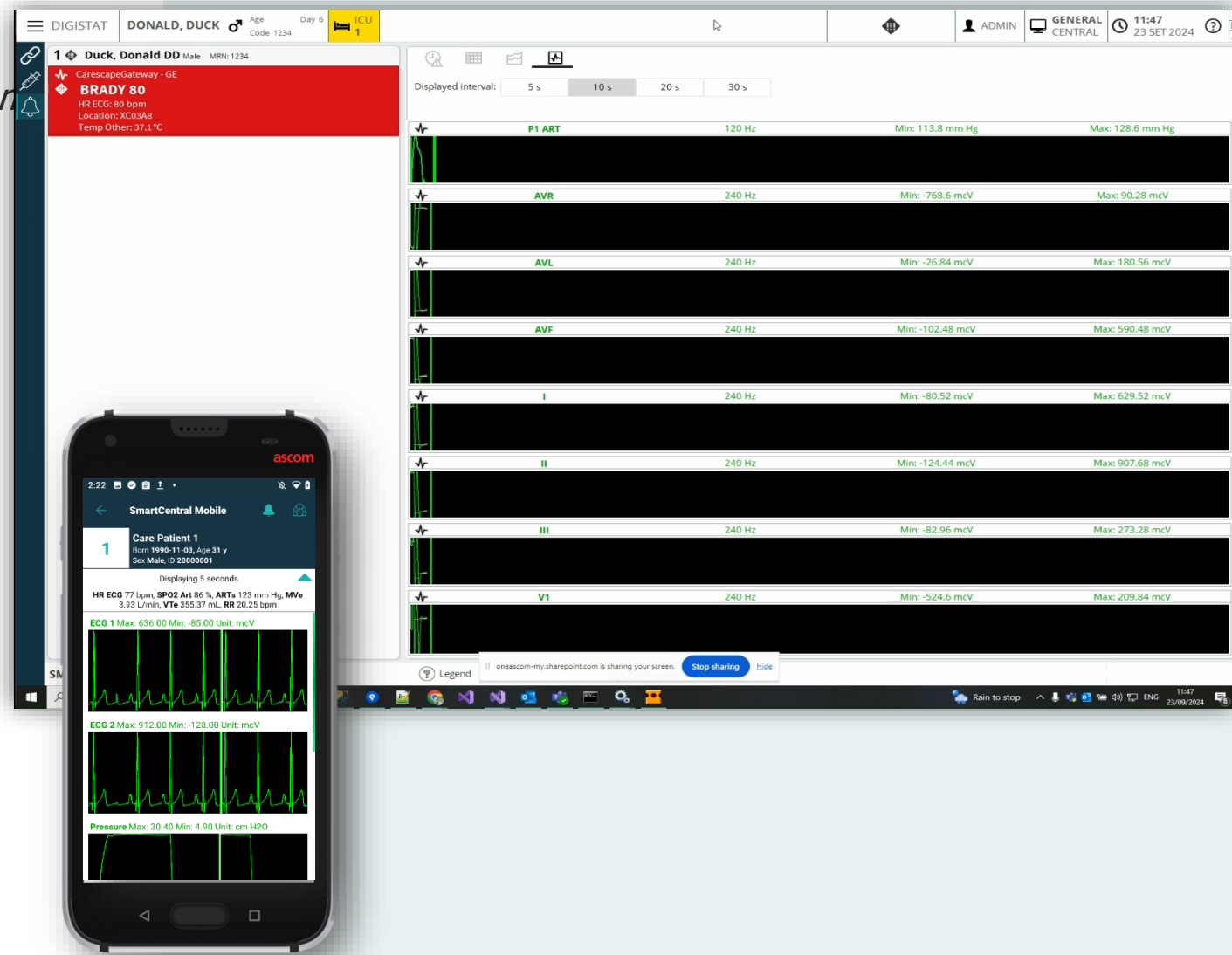
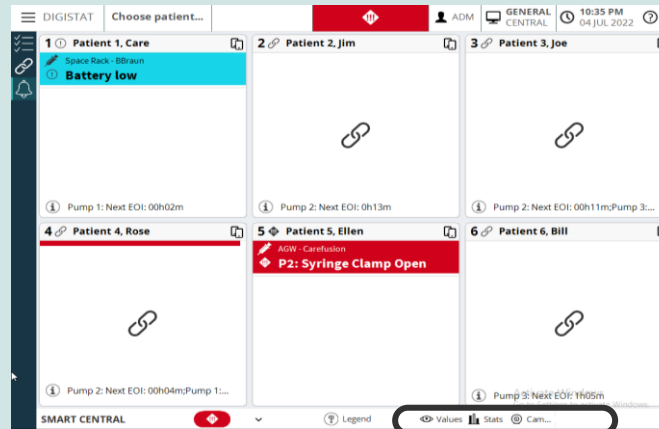
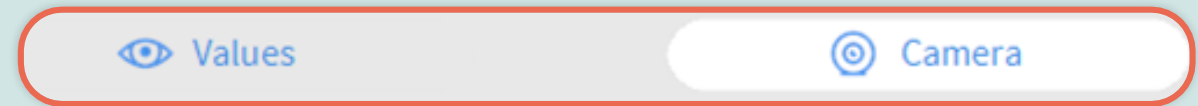
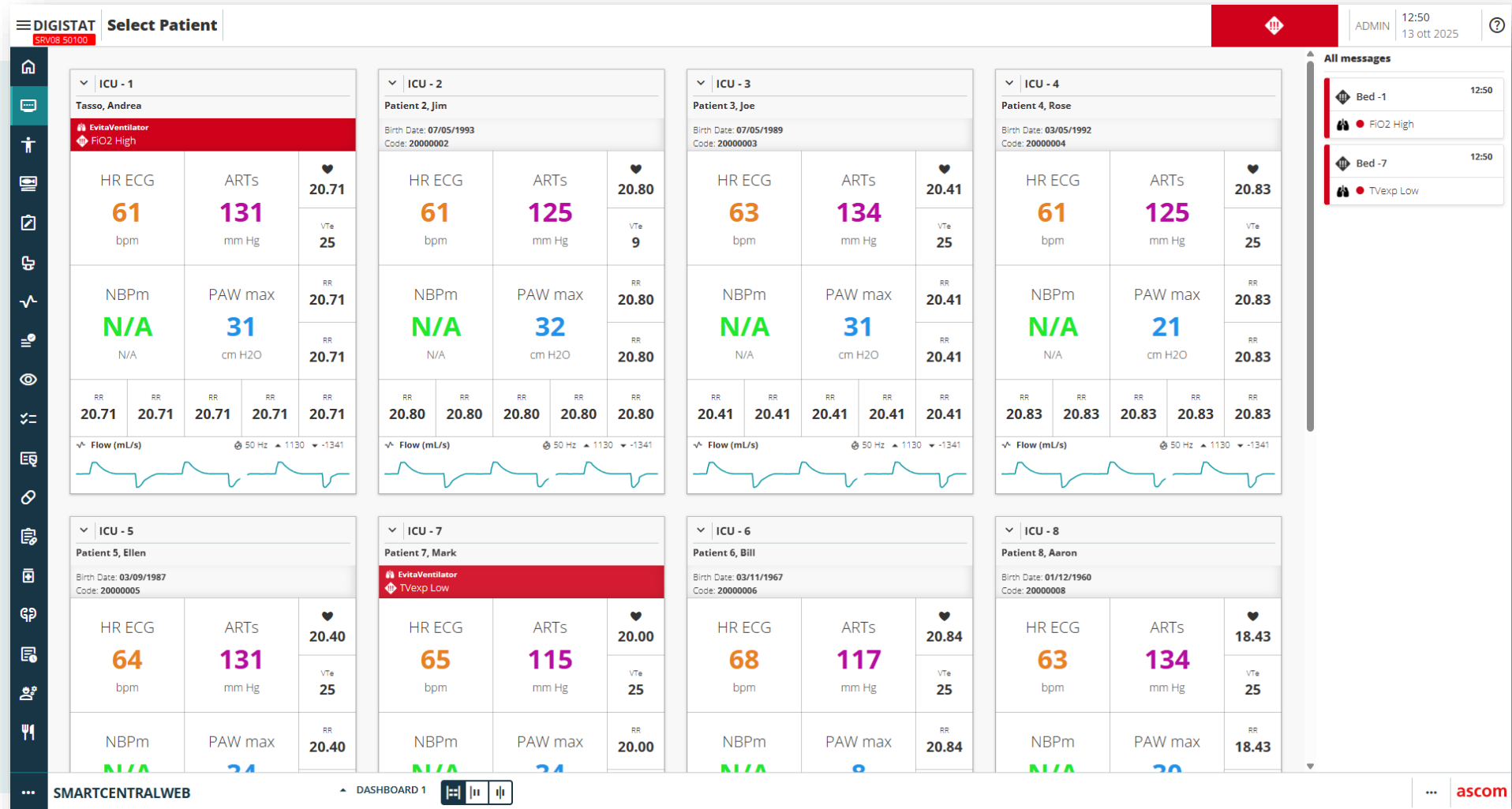
# Smart Central Web – Multipatient dashboard – still to be released

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Configure settings for rule Range Notifications for patient Doe, John

LOWER LIMITS      HIGHER LIMITS

Insert limits in ascending order, from left to right

> HR ECG (bpm) >    180

+ Add new parameter

Configure settings for rule Expression Notifications for patient Doe, John

CONDITIONAL   AND   OR   (   )      HR ECG>140 bpm AND Temp Core>38 °C AND SPO2 Art<93 %

HR ECG>140 bpm   AND   Temp Core>38 °C   AND   SPO2 Art<93 %

Notification Status:       Notification Level:

Notification Message:

+ Add new expression

Configure settings for rule EWS for patient Doe, John

	3	2	1	0	1	2	3				
Respir. Rate (bpm)	-2		-1	1	2	20	21	24	25		
Oxygen Saturation	91	92	93	94	95	96					
Temperature (°C)	35		35,1	36	36,1	38	38,1	39	39,1		
Systolic BP (mmHg)	90	91	100	101	110	111	219				
Heart Rate (bpm)	40		41	50	51	90	91	110	111	130	131

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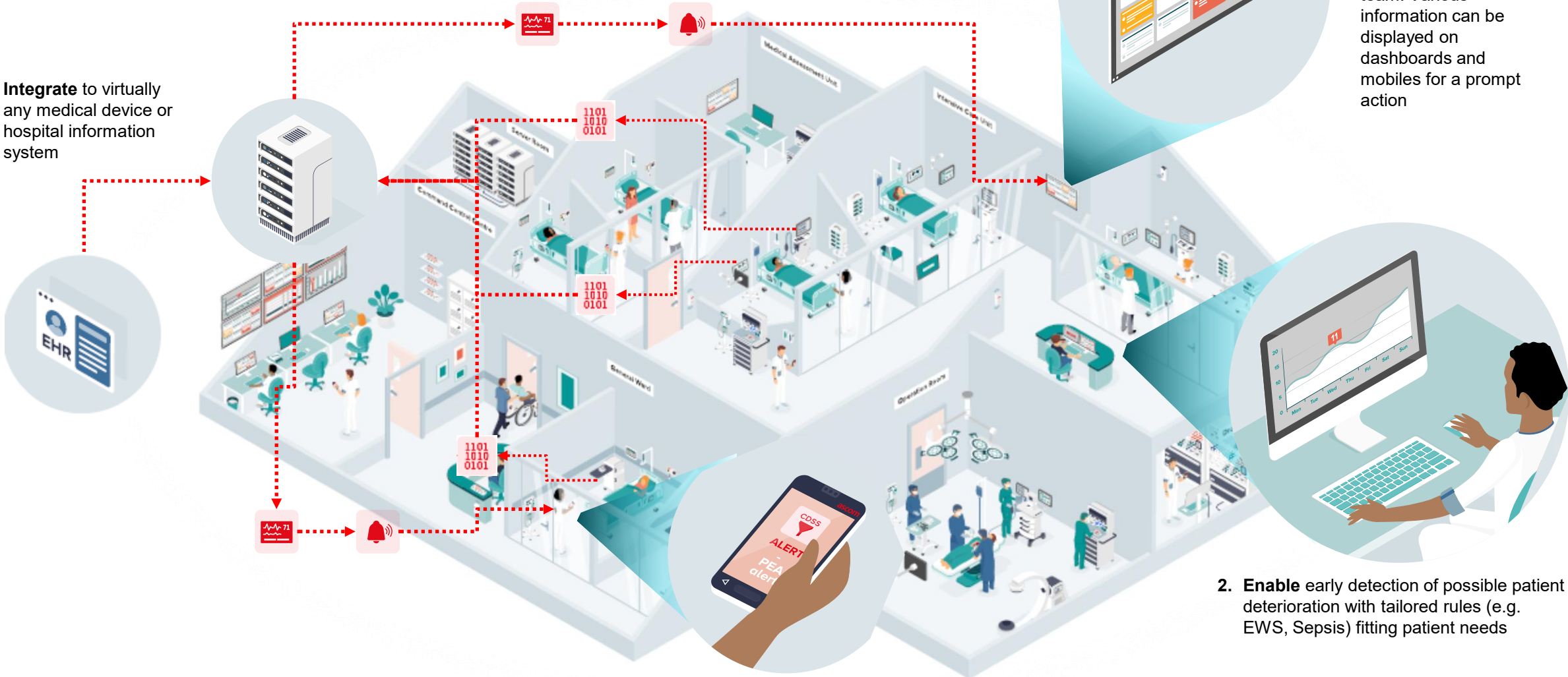
1. **Integrate** to virtually any medical device or hospital information system

3. **Distribute** notifications helps reducing delays in informing the care team. Various information can be displayed on dashboards and mobiles for a prompt action

2. **Enable** early detection of possible patient deterioration with tailored rules (e.g. EWS, Sepsis) fitting patient needs

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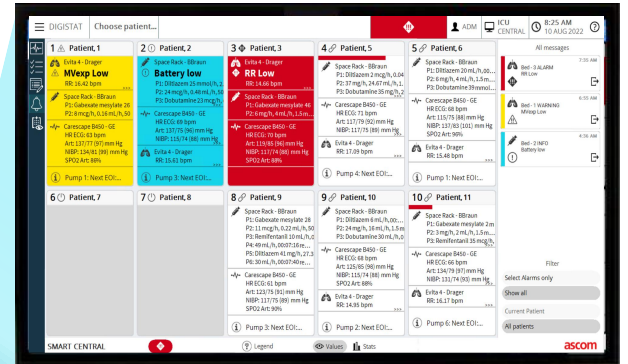
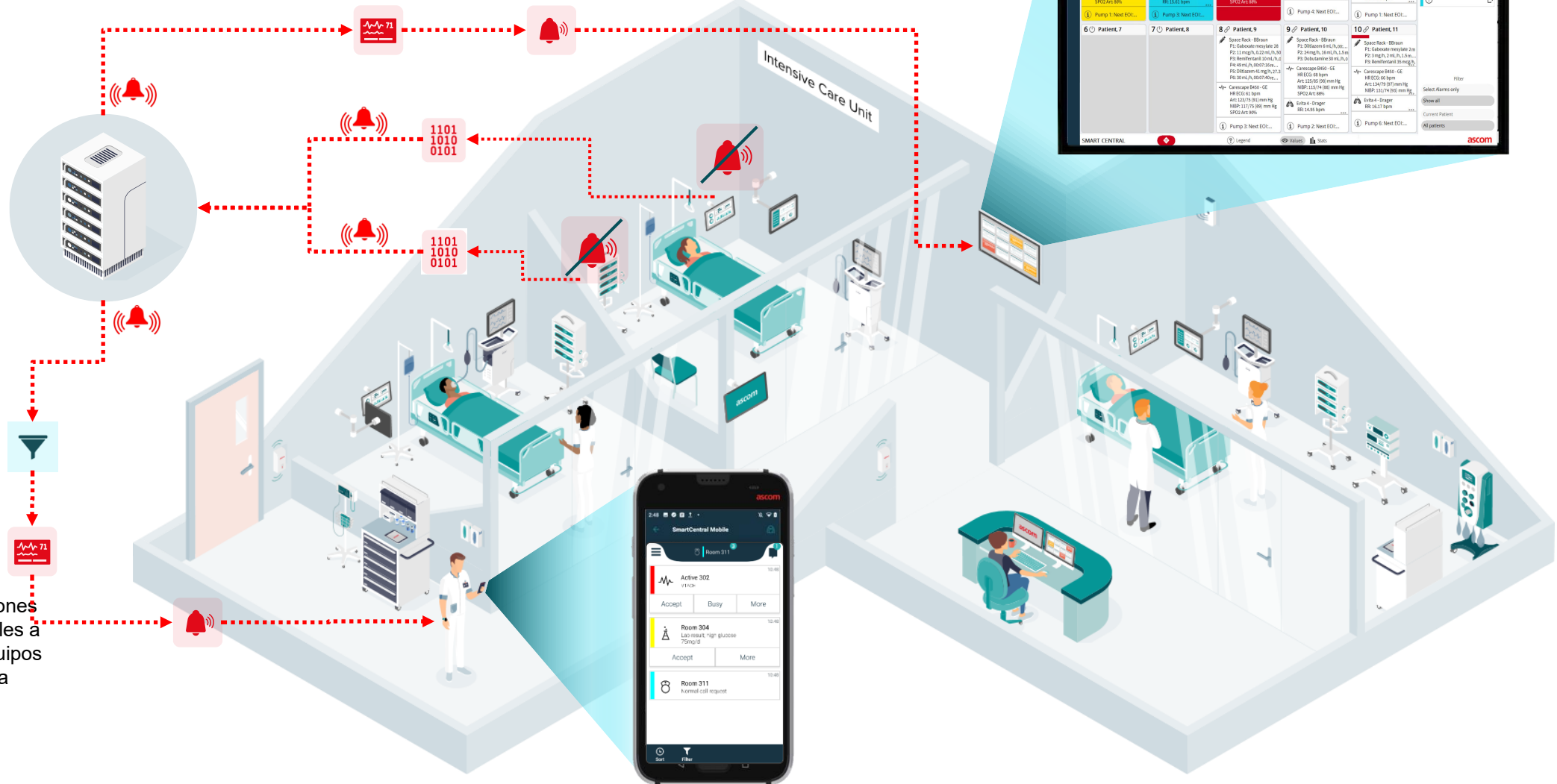
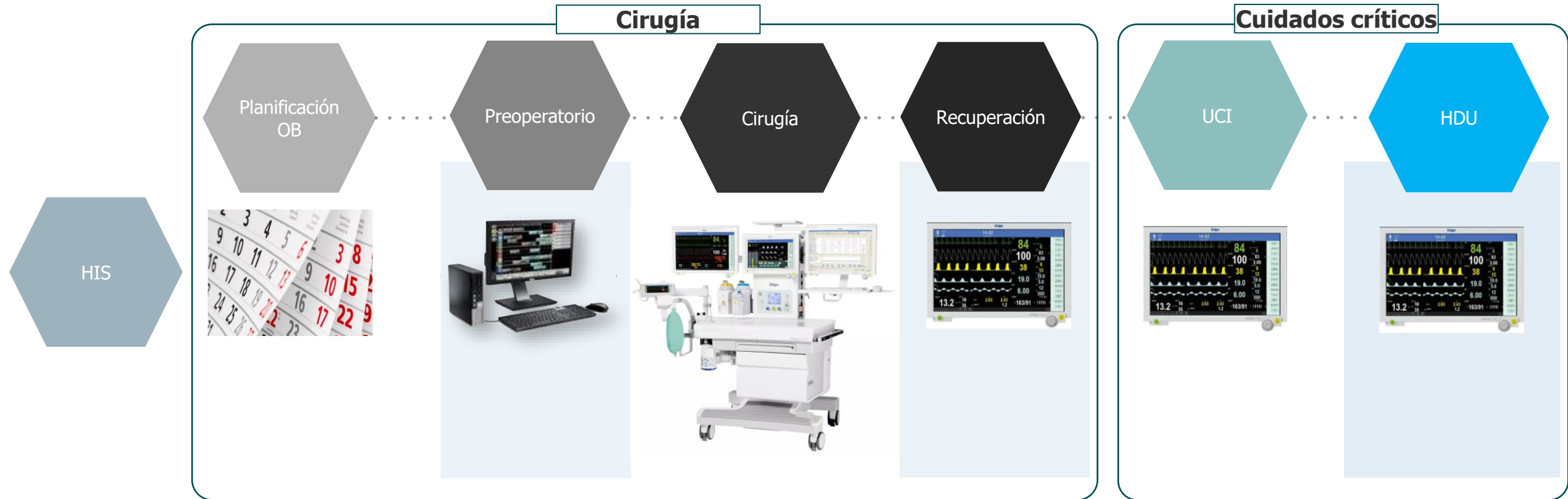
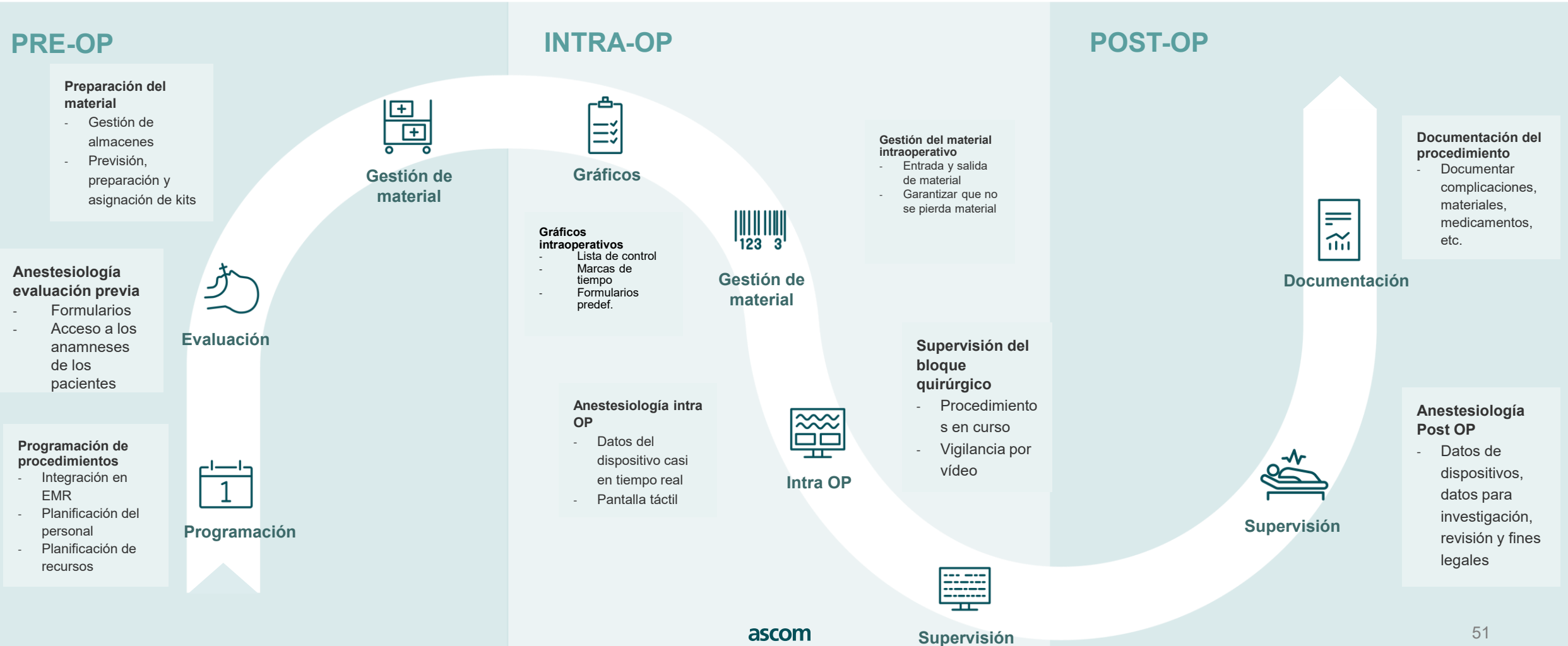
# Scheduling – Patient Card

The screenshot displays a web-based medical scheduling application. It features a sidebar with navigation icons (Home, SCHED, KITLIST, WPLSC, WPLT, WRS, WTK, WFM, FARM) and a main content area with multiple overlapping windows.

**Window 1: Patient Selection**  
 - Title: Selezione paziente...  
 - URL: itfco-vsrv02sao:8020/WOL/OperationLists/Result/  
 - Time: 23:10, 13 GIU 2023

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- KITLIST
- SCHED
- OPLIST
- WPDF
- WPLSC
- WRS
- WPLT
- WTK
- Home
- WFAR

- Unità Operative
- Cerca...
- CARDIOCHIRURGIA
  - CHIR MAXFAC
  - END.DIGEST.CH.
  - GIN.ONCOLOGICA
  - GINECOLOGIA
- Sale Operative
- CCH.A
  - E.A
  - O.1
  - O.2

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SlotManager ZOOM: Start Time End Time STAMPE Planning Wizard ANNULLA SALVA

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Sistema Cardiovascolare

# Material Management

*Supporting management of material*

**ascom**

# Material Management

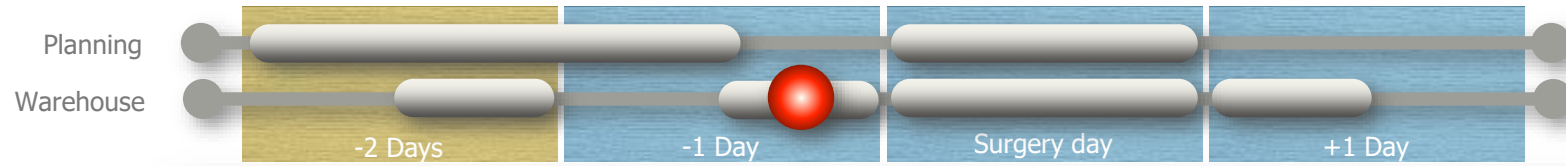
Supporting management of material

- *Material associated with procedures*
- *Material kits are prepared on time*
- *Kit delivered on time*
- *Local warehouse holding reduction*
- *Availability of material is shared between the surgical team*

# Kits of Material Preparation

Kits are usually prepared in the afternoon of the day before with a pick-up list

# Intraoperative Phase

*Supporting anesthetists activities*

# Operating Block

*Check-in workstation to support patient workflow*

# Operating Room

*2 workstations are needed for an optimal workflow*

# Intraoperative Workstations

*Anesthetist and nurse workstations for an optimal workflow*

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Home Verbal OPULIST SCHED Plan Oranj Pazien Stock Kit Online Valida Anesth Surgic

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▼ Checklist Operatorie  
Checklist Sign IN  
Checklist Time OUT  
Checklist Sign OUT

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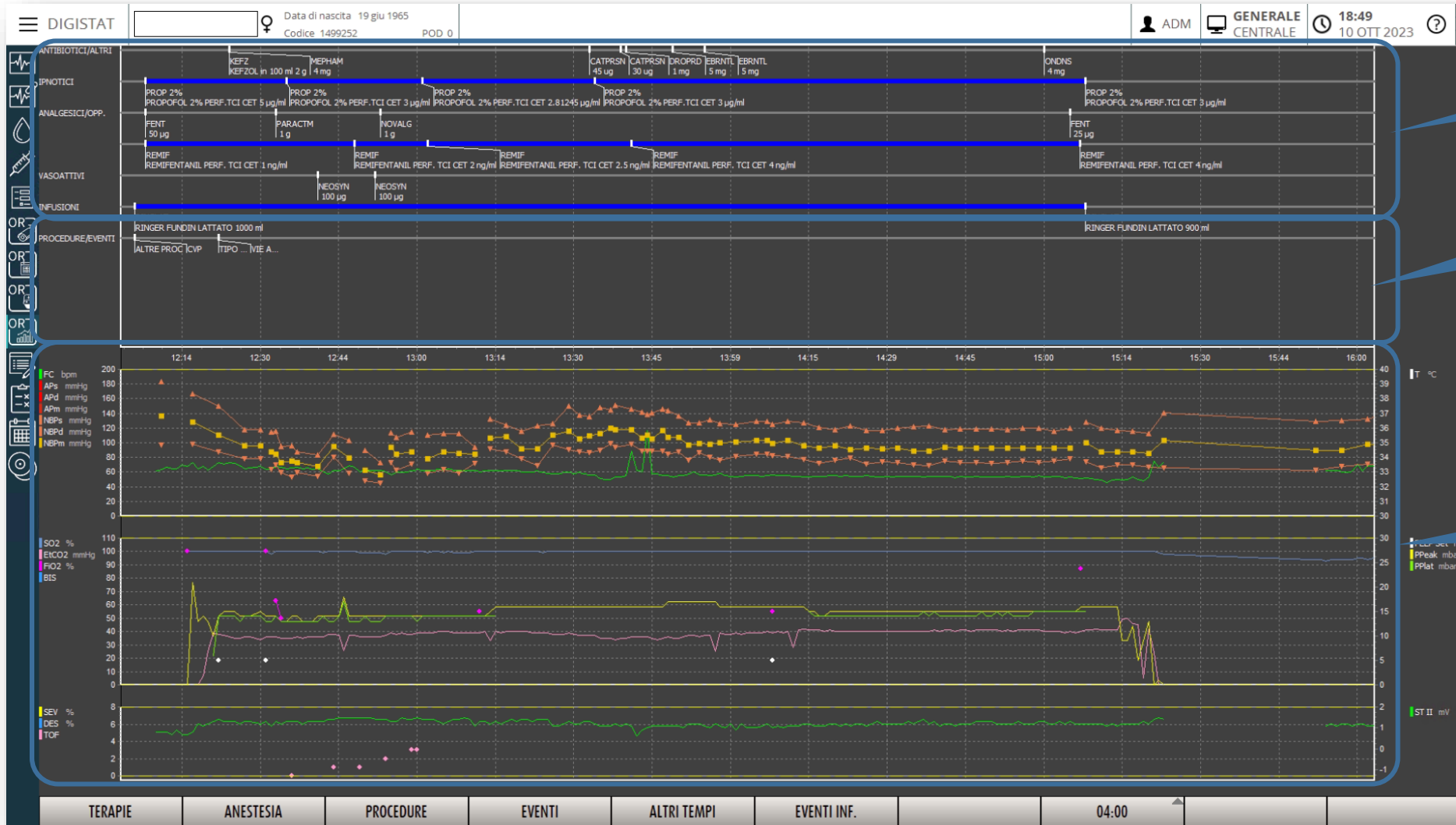
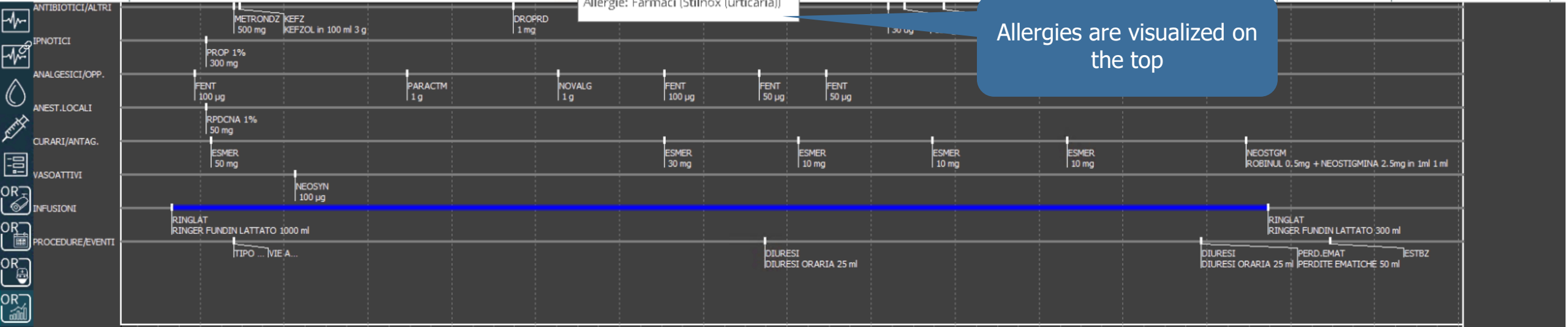
POD 0

Allergie: Farmaci (Stilnox (urticaria))

Allergies are visualized on the top

All medications are easily accessible from this link

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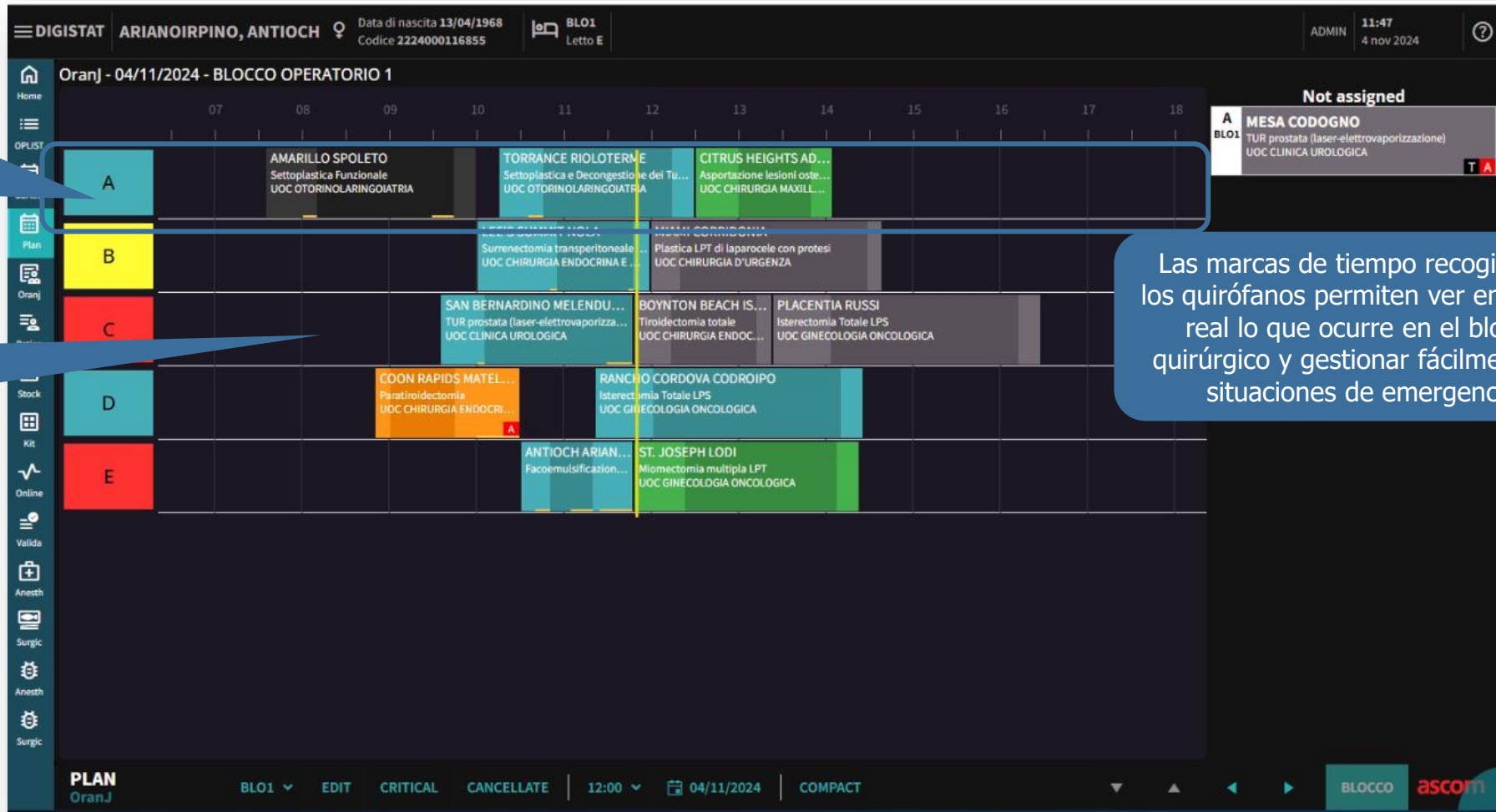
## OPERATING ROOM DOSSIER

SUSAN SMITH 16.07.1937 - Case No.: 0000000 - Intervention Date: 10.10.2023

<b>SUSAN SMITH - 86 years old</b>		<b>Sex: F</b>
Intervention date	10.10.2023	Operator AAA
Diagnosis	T81.3: DEHISCENCE OF OPERATIVE WOUND NOT ELSEWHERE CLASSIFIED; REVISION RIGHT HIP WOUND	
Proposed intervention	GENERIC EMERGENCY INTERVENTION REVISION RIGHT HIP INJURY. SIDE: NOT APPLICABLE	
Specialties	13- Orthopedic surgery	

### PREOPERATIVE REASSESSMENT

Creat. clearances (ml/h)	Weight (Kg)	Perspiratio (ml/Kg/h)	Preoperative fasting
Not calculated	74		YES
New events	NO		
ASA	ANESTHESIA TYPE.	SIGNED CONSENT	
3	General anesthesia	YES	
Suitability	IDONE		

### ANESTHESIA

ANESTHESIA Type: 2- Inhalation anesthesia;  
 AIRWAYS Device: 4- Oral intubation; Vent.in mask: 4 hands; Mask type: Intersurgical 3; ML type: I-Gel 4; Intub. operator: dri; Cormack-Lehane: I;  
 Depth cm: 20; Cuff pressure cmH2O: 30; Tube type: Normal endotracheal; Tube size: 7.0; (laryngeal mask not airtight intubated)

OPERATING STAFF	IN	OUT	INTERVENTION DATA
ANESTHETIST			Operating block <b>BO</b>
1 SURGEON			Operating room <b>HALL 2</b>
CLINICAL ASSISTANT			ANESTHESIOLOGICAL TIME <b>00:18</b>
ANE NURSE.	16:00		SURGICAL TIME <b>00:34</b>
ANE NURSE.		16:04	Postoperative care <b>1- Awakening Room</b>
SCRUB NURSE			
VOLANTE			
HOSPITALITY	16:30		

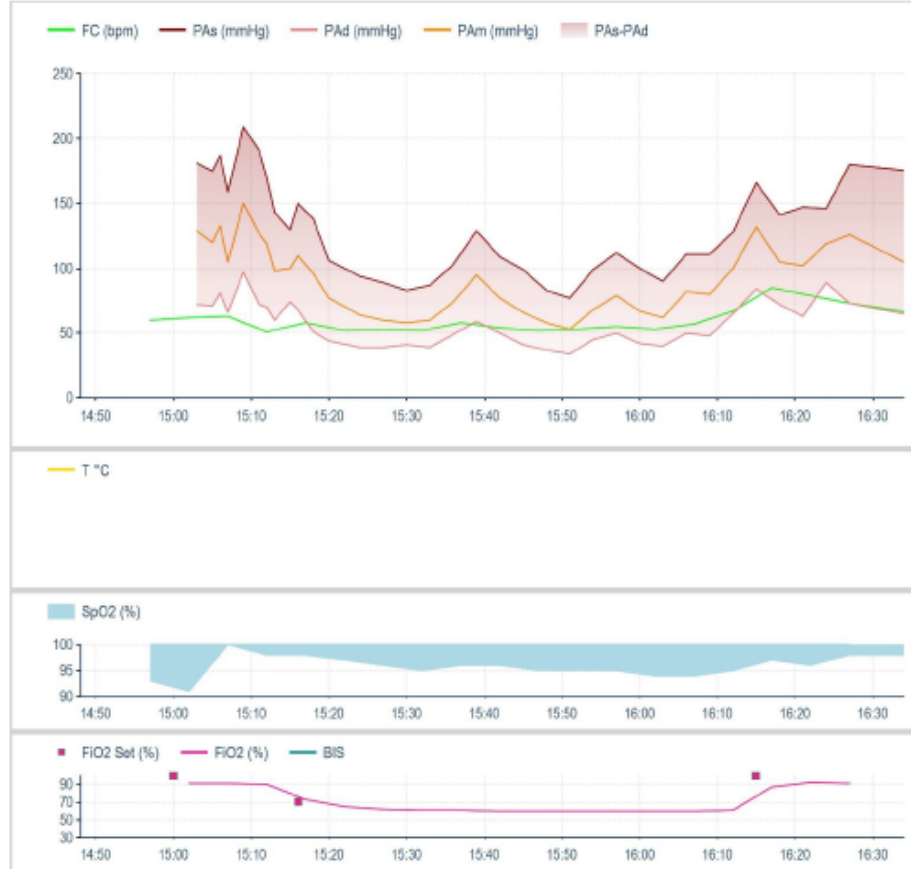
INTRAOPERATIVE BALANCE SHEET		
FLUID	FLUIDS IN	127
	Total FLUIDS	127
	TOTAL BUDGET	127

## OPERATING ROOM DOSSIER

SUSAN SMITH, 16.07.1937 - Case No.: 0000000 - Date of Operation: 10.10.2023

### DRUGS - TIMING

14:14	> PATIENT CALL
14:47	> INPUT BLOCK
14:48	> ANEST INTAKE
14:52	> HALL ENTRANCE.
15:00	START RINGER FUNDIN LACTATE 1000 ml
15:00	OTHER PROCEDURES Monitoring: ECG; NIBP; Sat; ETCO2; Temp °C: Superficial; Eye protection: YES; Heating pz: Upperbody;
15:00	PERIPHERAL VENOUS CATHETER In situ: YES; Access: Cubital SX; Type: G18/32mm green I.V. catheter;
15:05	NEOSYNEPHRINE BOLO 100 µg
15:05	EFEDRINE 6 mg
15:05	FENTANYL BOLO 50 µg
15:05	PROPOFOL 1% BOLO 150 mg
15:05	> START INDUZ/PROC ANEST
15:12	ESMERON BOLO 40 mg
15:13	FENTANYL BOLO 50 µg
15:23	START PREP. INTERVENTION (PATIENT POSITIONING) Position: Supine; Delt.Supine : Open arms;
15:23	> END INDUZ/PROC ANEST
15:23	> START PREP. INT. (DISINF.)
15:34	> TIME OUT
15:34	> ENGRAVING/START INT.
15:38	FENTANYL BOLO 50 µg
15:50	EFEDRINE 10 mg
15:51	CO AMOXI 2.2 GR 2.2 g
16:03	EFEDRIN 10 mg
16:08	> SUTURE/END INT.
16:09	ROBINUL 0.5MG + NEOSTIGMINE 2.5MG IN 1ML 1 ml
16:18	PROPOFOL 1% BOLO 50 mg
16:18	VENTOLIN 200 INAL. 100 MCG/PUFF Puff: 0;
16:18	> END PROC. OPER.
16:26	ESTUBATION
16:26	ANESTHESIST'S NOTE (bronchospasm before extubation with desat at 78% resolved with AG reapproximation and ventolin)
16:26	> END OF AWAKENING.

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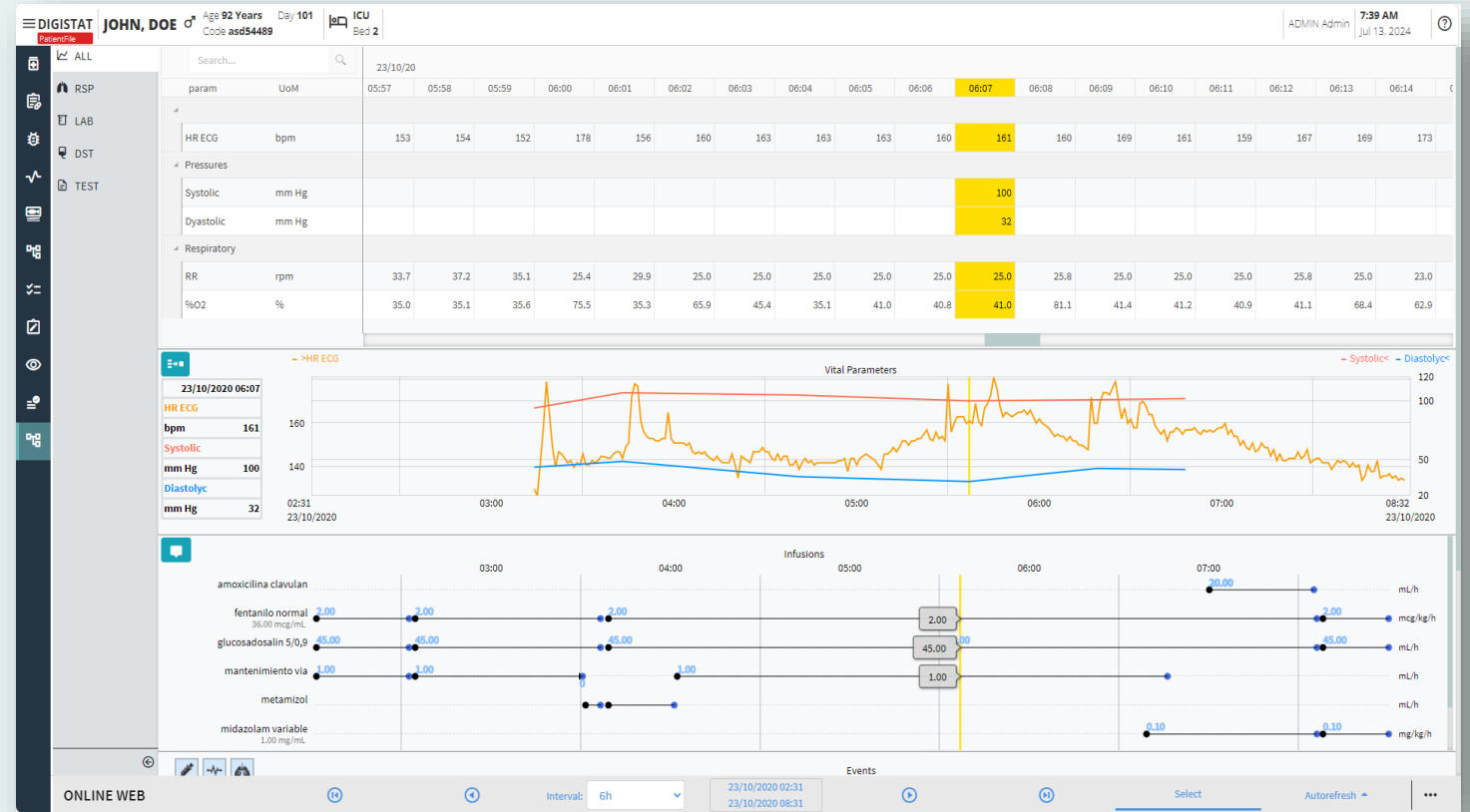
*Managing care cycle*

# Patient Assessment

*Structured workflow to support patient assessment during the care*

- Clinical workflow implemented to support the assessment of the patient status since the admission visit
- Specific assessments to support nurses and physicians workflows
- Configurability
- Sections can be collapsed for an easy evaluation
- Supporting variables of different kinds (e.g. Text, numeric, list, multiple choices, ...)
- Free text available when required
- Definition of mandatory fields
- Data can also be automatically imported from third-party systems
- Possibility to choose between different printouts

Patient admission page with different kind of variables

Workflow supporting patient assessment

# Scoring

*Easy management of ICU scores*

- Easy score evaluation with a tabular view of results
- Easy configuration
- Predefined scores for different kind of ICUs already available
- Review of historical data collected
- Results can also be visible in a graphical way in the Online module

Specific score (e.g. SAPS II) under evaluation

List of scores active for the unit

# Dashboard

*Visualize any kind of structured data in a dedicated dashboard*

- Configurable dashboard to visualize data acquired from medical devices, systems (e.g. Lab) and manual input
- Specific dashboards can be configured according to the need
- Easy validation of data acquired from medical devices
- Same dashboard can also be available on android devices
- Each dashboard can be customized as decided, based on the combination of different widgets
- Data can be visualized in a grafical or tabular format

**Different dashboards**

**Vital parameters Tabular widget**

param	UoM	05:57	05:58	05:59	06:00	06:01	06:02	06:03	06:04	06:05	06:06	06:07	06:08	06:09	06:10	06:11	06:12	06:13	06:14
HR ECG	bpm	153	154	152	178	156	160	163	163	163	160	161	160	169	161	159	167	169	173
Pressures																			
Systolic	mm Hg																		
Dyastolic	mm Hg																		
Respiratory																			
RR	rpm	33.7	37.2	35.1	25.4	29.9	25.0	25.0	25.0	25.0	25.0	25.0	25.8	25.0	25.0	25.0	25.8	25.0	23.0
%O2	%	35.0	35.1	35.6	75.5	35.3	65.9	45.4	35.1	41.0	40.8	41.0	81.1	41.4	41.2	40.9	41.1	68.4	62.9

**Vital parameters Graphical widget**

**Infusion widget**

Drug	Unit	Value	Unit
amoxicilina clavulan	mL/h	20.00	mL/h
fentanilo normal	mcg/kg/h	2.00	mcg/kg/h
glucosadosalin 5/0,9	mL/h	45.00	mL/h
mantenimiento via	mL/h	4.00	mL/h
metamizol	mL/h	1.00	mL/h
midazolam variable	mg/kg/h	0.10	mg/kg/h

# Laboratory widget

Allowing easy charting of laboratory values together with vital parameters (PDMS focus)

The screenshot displays a laboratory data interface with two main data tables. The top table, titled "Laboratorio Agg1", shows hematology parameters for "Sangue intero" on 18/09/22 and 19/09/22. The bottom table, titled "test no AGREG", shows glucose and coombs test results for the same dates. A callout box on the left points to the parameter names, and another callout box on the right points to the time-based data columns.

param	UoM	Range	18/09/22				19/09/22						
			06:00	08:00	15:00	17:00	06:00	06:00	06:00	07:00	08:00	16:00	16:00
EMOCROMO-Neutrofili-SI		2 - 8	8.06		9.01	17.89	8.37	10.24	15.07	8.97		13.51	9.85
EMOCROMO-Linfociti-SI		1.5 - 4	1.71		0.72	0.77	1.75	0.76	0.8	1.8		1.3	0.23
EMOCROMO-Monociti-SI		0.1 - 1	1.13		0.27	0.77	1.28	1.14	1.04	1.17		1.2	1.23
EMOCROMO-Eosinofili-SI		0.1 - 0.5	0.33		0.01	0.18	0.01	0.24	0.25	0.33		0.32	0.19
EMOCROMO-Basofili-SI		0 - 0.2	0.04		0.01	0.02	0	0.02	0.05	0.02		0.02	0.06
EMOCROMO-Mielociti-SI			N.V.		N.V.	N.V.	N.V.	N.V.	N.V.	N.V.		N.V.	N.V.
EMOCROMO-Promielociti-SI			N.V.		N.V.	N.V.	N.V.	N.V.	N.V.	N.V.		N.V.	N.V.
EMOCROMO-Blasti-SI			N.V.		N.V.	N.V.	N.V.	N.V.	N.V.	N.V.		N.V.	N.V.
EMOCROMO-Eritroblasti-SI			N.V.		N.V.	N.V.	N.V.	N.V.	N.V.	N.V.		N.V.	N.V.

param	UoM	Range	18/09/22		19/09/22			
			06:00	15:00	06:00	06:00	06:00	08:00
GLUCOSIO-GLUCOSIO-SIE		74 - 100	88	386	270	159	471	126
Interpretazione-Interpre...								
COOMBS DIRETTO-COO...								

Different parameters

Values according to date and time

# Microbiology widget

Allowing easy charting of microbiology values together with vital parameters (PDMS focus)

The screenshot displays the 'Microbiologia' widget interface. On the left, a sidebar lists various sample types: MICRO YCT, MICRO Mob, LAB WEB, test new, longRangeX, G&C, GRID ONLY, LAB MOBILE, DEMO, MOBI, and Sql Chart. The main area is a grid with columns for dates (24/08/..., 29/08/..., 31/08/..., 06/09/2022) and rows for sample types: Exam (code), Tampone rettale (TAMRE), Tampone nasale (TN), Broncoaspirato (BAS), and Urina da mitto intermedio (URCOLT). Each cell contains a sample ID and a status icon (e.g., a red bug for positive, a green bug for negative). A teal callout box labeled 'Sample type' points to the 'Tampone rettale (TAMRE)' row. Another teal callout box labeled 'Antibiogram' points to the 'Tampone rettale (screening o sorveglianza)' popup. The popup for 'Tampone rettale - 31/8/2022, 09:01:15 (Id:80308342)' shows a detailed result including a text note, a value of 'KLEPNE', and a table of antibiotic resistance results. The popup for 'Tampone nasale - 31/8/2022, 09:01:15 (Id:80308341)' shows a text result of 'Esame culturale Negativo' and a code of 'ECOLT'.

Sample Type	24/08/...	29/08/...	31/08/...	06/09/2022
Exam (code)	10:09	11:29	11:01	11:00
Tampone rettale (TAMRE)	80307865		80308342	
Tampone nasale (TN)	80307864		80308341	
Broncoaspirato (BAS)		80308189		
Urina da mitto intermedio (URCOLT)				80308796

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Text Esame culturale  
Value Negativo  
Status  
Abnormal  
Code ECOLT

# Clinical Diary

*Managing the clinical diary in a flexible way*

- Free text for any kind of clinical note
- Different colors according to the profile
- Possibility to define templates
- Possibility to tag clinical notes in a specific way for a fast research
- Clinical notes can also be inputted using the mobile
- Possibility to import images
- Clinical diary can be filled in automatically also of structured assessments made during the patient assessment

The screenshot shows a web interface for a clinical diary. On the left, there is a 'FILTERS' sidebar with a list of categories: ALL (checked), Burns (red), Cardiology (pink), Clinical Nurse Speci... (blue), Dermatology (orange), Diagnostic Imaging (brown), and Dietit... (light pink). Below the filters are options for 'Show a...', 'Calendar', 'Authors', 'Tags', 'Med...', 'Sta...', 'Rel...', and 'For...'. The main content area displays three entries, each with a colored header bar: green for 'Occupational Therapist', blue for 'Clinical Nurse Specialists', and pink for 'Cardiology'. Each entry contains placeholder text and is attributed to 'ADMIN, ADMIN - ADM'. At the bottom, there is a navigation bar with buttons for 'DIARY WEB', 'NEW', 'HISTORY', 'EDIT', 'DELETE', 'SEARCH', 'PRINT', and 'REFRESH'. Three callout boxes provide additional information: one points to the filter list with the text 'Different color for different', another points to the colored entry headers with 'Different color code according to the profile of the user', and a third points to the list of entries with 'List of diary notes related to the patient'.

# Clinical Diary – writing a note

*A note can be written both on the web module or with the mobile application*

The screenshot displays the 'Plastic' department interface. On the left, a sidebar lists various user profiles with color-coded circles: Gastrology, Joint Therapist, Maxillo Facial, Nurse, Obstetrics and Gynaecology, Occupational Therapist, Oncology, Other, Pharmacists, Physician, Physiotherapist, Plastic (highlighted in red), Psychiatry, Psychology, and Renal. The main area features a dropdown menu for 'Observations, assessments and findings', a rich text editor with 'Format' options (B, I, U, abc, bulleted list, numbered list, link, unlink, indent, outdent, bold, italic, underline, strikethrough), and buttons for 'TEMPLATE' and 'PHRASES'. Below the editor is a text area containing placeholder text. At the bottom, there are image upload icons, a tag selection panel with categories like Communication, Consultant, Diagnosis, Family, ICNARC, Imaging, Infection, Invasive Lines, Medications, and Nutrition, and a time/date selector showing 5:59 on 2/8/20. At the bottom right, there are 'DRAFT', 'Save', and 'Cancel' buttons.

Different color code according to the profile of the user

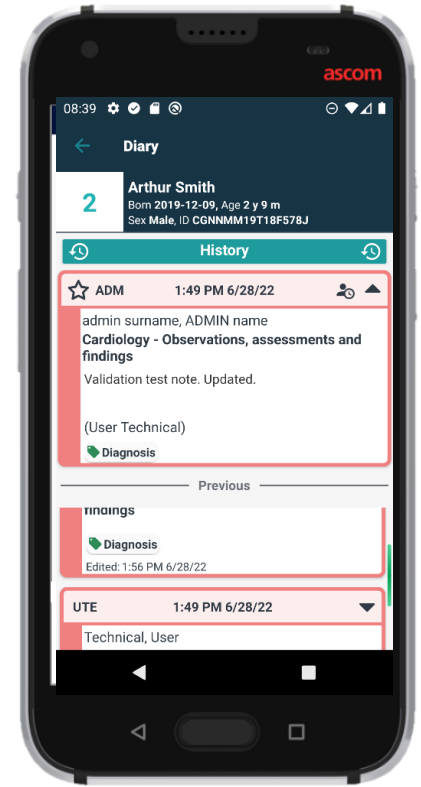
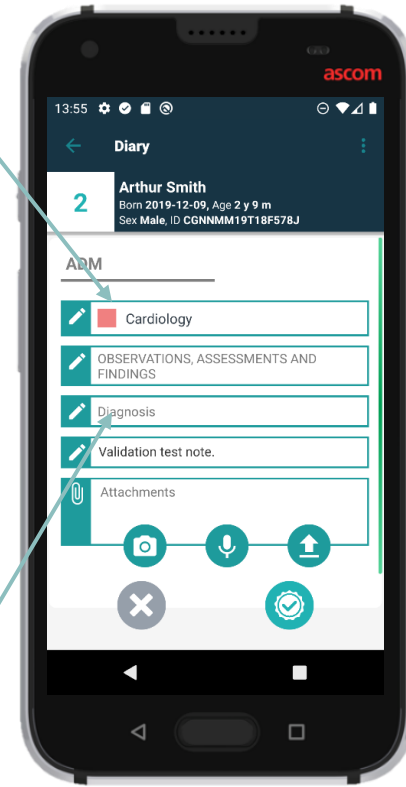
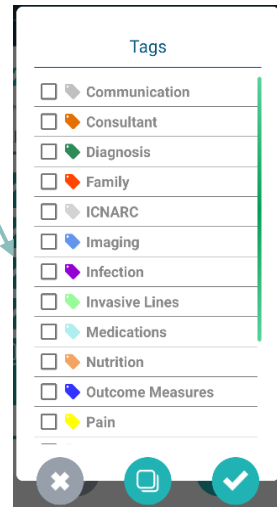
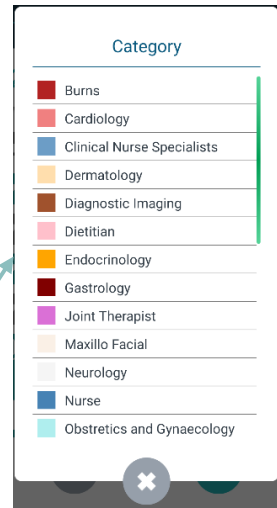
Possibility to create pre-defined template or phrases

Possibility to attach images or other docs

Tags that can be used for a note, for a faster grouping or search

# Diary

Writing a note on the mobile

# Fluid Balance

*Manage fluids during care of acute patients*

- Both tabular and graphical interfaces can be chosen according to the need
- Data can be acquired automatically from medical devices or inputted manually
- Dark mode also possible according to configuration
- Possibility to add notes and define targets
- Weight of the patient can be displayed in each column
- Data can be shown in accruing mode
- Possibility to configure
  - Time interval of interest
  - Switch time that define the clinical day
  - Weight mode available
  - Fluids in/out

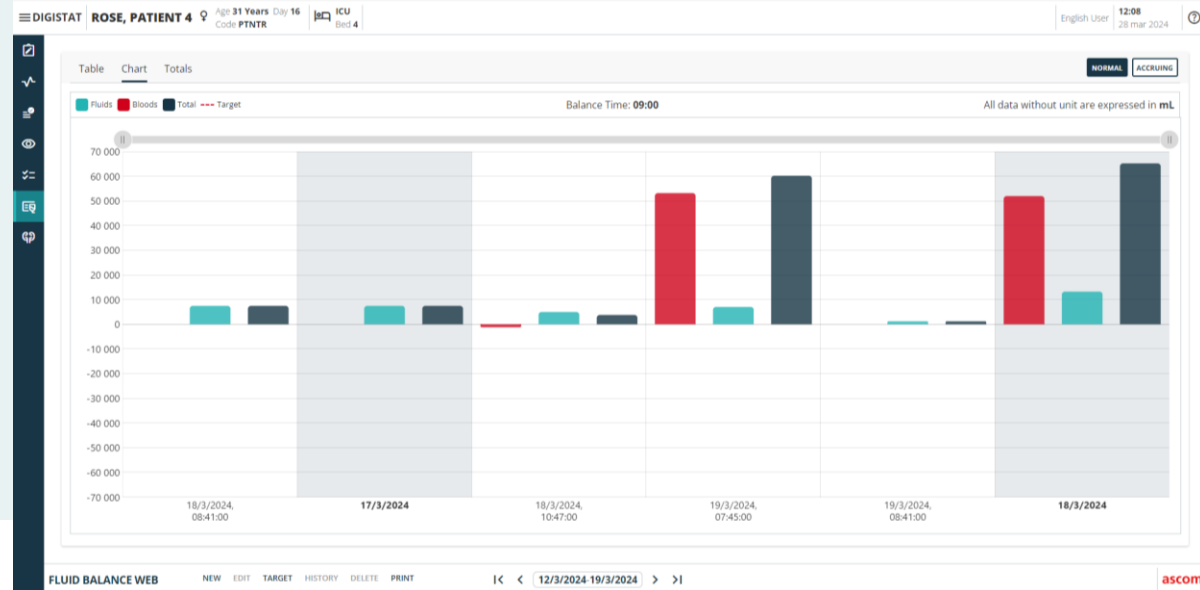
DIGISTAT ROSE, PATIENT 4 Age 31 Years Day 16 ICU Bed 4 English User 12:03 28 mar 2024

Table Chart Totals NORMAL ACCRUING

Target Last Edit Fluids Bloods Incoming Outgoing From Pump Weight Balance Time: 09:00 All data without unit are expressed in mL

Date	18/3/2024	17/3/2024	18/3/2024	19/3/2024	18/3/2024
Time	08:41	Daily	10:47	07:45	08:41
Target					
TOTAL BALANCE	7477.460	7477.460	50	70	1211.930
FLUID TOTAL	7477.460	7477.460	50	70	1211.930
FLUID IN	7477.460	7477.460	50	70	1211.930
FLUID OUT	0	0	0	0	0
BLOOD TOTAL	0	0	0	0	0
BLOOD IN	0	0	0	0	0
BLOOD OUT	0	0	0	0	0
Diltiazem (infusion)	1402.380	1402.380			245.010
Dibutamine (infusion)	1522.960	1522.960			267.780
Gabexate mesylate (infusion)	1458.010	1458.010			236.190
INFUSION (infusion)	3094.110	3094.110			462.950
EXTRA IN			50	70	
Fluid IN					120
DIURESIS					
DRAINAGES					
EXTRA OUT					
PERSPIRATIO					
Fluid OUT					
Blood IN					
Blood OUT					
Last Edit User	ADM		ADM	ADM	ADM

FLUID BALANCE WEB NEW EDIT TARGET HISTORY DELETE PRINT |< < 12/3/2024-19/3/2024 > >| ascom

# Prescription

*Managing care cycle*

# Therapy prescription

*Manage proper therapy prescription for each patient*

- Patient therapy overview with prescription workflow, therapy cycle update, time plan
- Prescription as commercial product or active principle
- Predefined prescriptions available
- Medications can be grouped for an easy search
- Execution timeline with drugs administration workflow
- Customizable catalog (active moieties, products, forms, routes and more)
- Native support for external sources like external Drug Database
- Drugs interaction support
- Allergies support

The screenshot displays the DIGISTAT interface for patient JIM, PATIENT 2. The top navigation bar includes the patient's name, age (30 Years), and day (35). The main content area shows a list of active prescriptions, grouped by class. The 'ANALGESICS' group includes PARACETAMOLO (500 mg ORALE, if temperature > 38.5 °C). The 'ANTIBIOTICS' group includes AMOXICILLINA (10 mg ORALE, once a day) and CEFATRIZINA (10 mg in 100 ml at 200 ml/h, once a day). The 'NONSTEROIDAL ANTI-INFLAMMATORY' group includes IBUPROFENE (300 mg ORALE, 2 times a day). Other prescriptions include AMOXICILLIN SODIUM\*1F 1G/10ML and OSSICODONE (OXICODONE). The interface features a sidebar for filters, a top navigation bar with 'REVIEWED' and 'NEXT UPDATE' buttons, and a bottom navigation bar with 'THERAPY WEB PRESCRIPTION', 'NEW', 'OVERVIEW', and 'UPDATE' options. Callouts highlight 'Visualization per groups' (pointing to the filter menu) and 'Therapy reviewal time' (pointing to the top right area).

# Therapy Web - Central

≡ DIGISTAT | **Select Patient** | ADM | 17:31 | 16 Oct 2024 | ?

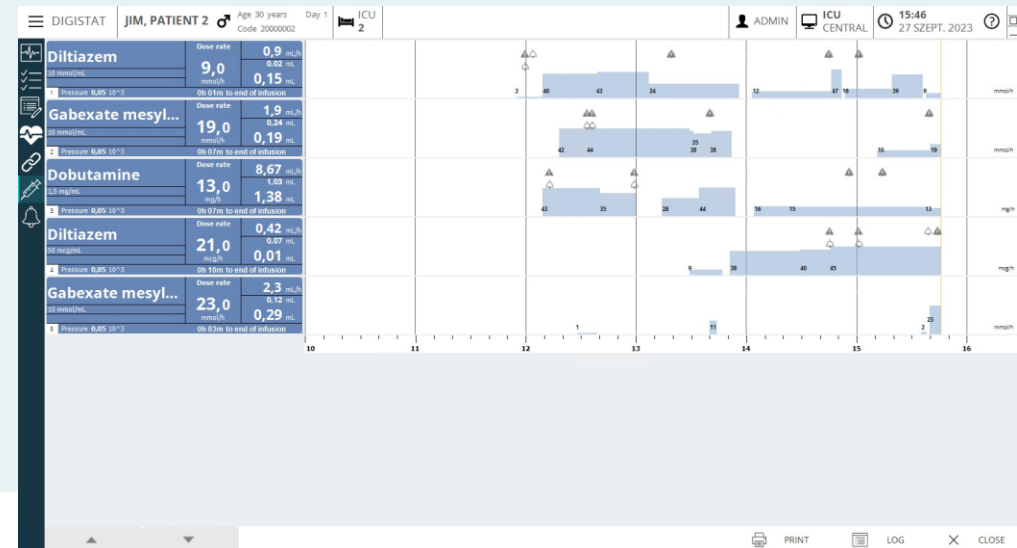
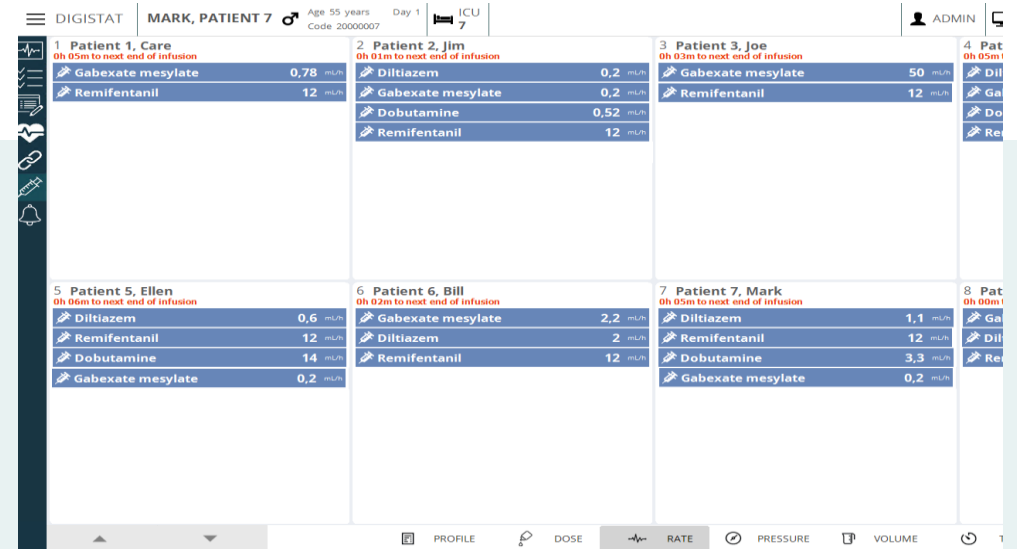
<p>ICU 1 <b>Care, Patient 1</b> 20000001 DOB: 11/05/1998</p> <p>REVIEWED 5 hours ago</p> <p>✓ <b>READY</b> 1 📄 <b>PENDING</b> 1</p>	<p>ICU 2 <b>Jim, Patient 2</b> 20000002 DOB: 11/05/1998</p> <p>REVIEWED 5 hours ago</p> <p>✓ <b>READY</b> 2 🔄 <b>RUNNING</b> 1</p>	<p>ICU 3 <b>Joe, Patient 3</b> 20000003 DOB: 11/05/1998</p> <p>REVIEWED 5 hours ago</p> <p>No active prescriptions</p>	<p>ICU 4 <b>Rose, Patient 4</b> 20000004 DOB: 11/05/1998</p> <p>REVIEWED 2 days ago</p> <p>📄 <b>PENDING</b> 2</p>	<p>ICU 5 <b>Ellen, Patient 5</b> 20000005 DOB: 11/05/1998</p> <p>REVIEWED 2 days ago</p> <p>🔔 <b>OVERDUE</b> 2</p>	<p>ICU 6 <b>Bill, Patient 6</b> 20000006 DOB: 11/05/1998</p> <p>REVIEWED yesterday</p> <p>🔔 <b>OVERDUE</b> 1</p>
<p>ICU 7 <b>Mark, Patient 7</b> 20000007 DOB: 11/05/1998</p> <p>REVIEWED yesterday</p> <p>🔔 <b>OVERDUE</b> 2</p>	<p>ICU 8 <b>Aaron, Patient 8</b> 20000008 DOB: 11/05/1998</p> <p>REVIEWED 8 days ago</p> <p>🔔 <b>OVERDUE</b> 2</p>	<p>ICU 9 <b>Bianchi, Maria</b> 11233 DOB: 11/05/1998</p> <p>REVIEWED --</p> <p>No active prescriptions</p>	<p>ICU 10 <b>Empty bed</b></p>	<p>Neurologia <b>Rossi, Mario</b> Bed 1 1111 DOB: 11/05/1998</p> <p>REVIEWED --</p> <p>No active prescriptions</p>	<p>Neurologia <b>Verdi, Giovanna</b> Bed 2 2998 DOB: 11/05/1998</p> <p>REVIEWED --</p> <p>No active prescriptions</p>

THERAPY WEB CENTRAL | Patient name | Bed name | OVERDUE | RUNNING | READY | PENDING | REVIEWED | PATIENT NAME | DEFAULT

# Infusion central

*Provides an overview of infusion pumps in the unit*

- Support nurses managing alarms
- Collect and record infusion information and events
- Provide statistics about pump usage and drug protocols
- Transmit pumps information and receives patient demographics from the Hospital Information System

# Patient Summary

Providing overview on patient status

- Customizable page to provide a patient overview according to needs (what visible here beside is just an example of what can be done)
- Multiple pages can be created according to needs, so to focus on different aspects of patient care
- Widget approach

The screenshot displays the Ascom Digistat patient summary for Kiefer, Sutherland M. The interface is organized into several key sections:

- Real time parameters:** Shows HR (bpm) at 61, RR (breath/min) at 19.22, and VTe (mL) at 55, all updated 9 seconds ago.
- Devices on the patient:** A table listing various medical devices such as Venous access, Abdominal drainage, Nerve block, Balloon Gastrostomy, and Chest drainage, along with their locations and types.
- Vital parameters dashboard:** Contains two line graphs: 'Circulatory' showing HR ECG, NBP, and SPO2 Art, and 'Respiratory' showing RR, SPO2, and SPO2 Art over time.
- Fluid Balance:** A bar chart showing fluid intake and output over a 24-hour period.
- Diary:** A section for medical notes, including a 'Diary notes' table with columns for date, department, plan, and notes.

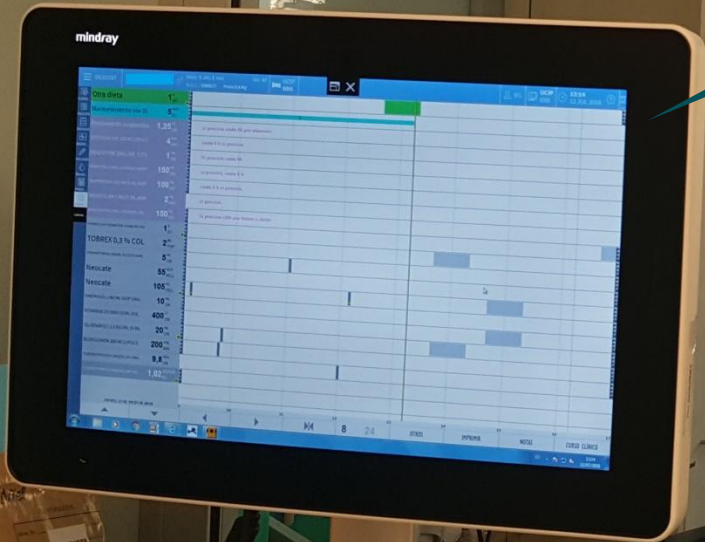
# Execution

*Managing care cycle*

5

Digistat running inside 3<sup>rd</sup> party monitoring systems solutions

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# Therapy execution

*Manage proper therapy administration for each patient*

- Prescriptions to execute can be grouped per typology according to needs
- Different color code to visualize typology of the prescription
- Clear indication of current timing
- Developed for an easy use with touchscreen at bed side

The screenshot displays the DIGISTAT interface for patient JIM, PATIENT 2 (Age 30 Years, Day 351, Code 20000002). The interface shows a timeline for therapy execution on Fri 12 April, from 08:00 to 15:00. The current time is 11:22 on 12 Apr 2024. The interface includes a navigation bar with 'OVERDUE', 'READY', and 'PENDING' indicators. The main area lists several prescriptions with color-coded bars indicating their status and timing:

- AMOXICILLIN SODIUM\*1F 1G/10ML — SOLUZIONE INIETTABILE** (Red bar): DOSE: 10 ml at 5 ml/h — ENDOARTERIOSA DILUITO — 12/04/2024, 06:00
- IBUPROFENE — COMPRESSE** (Green bar): DOSE: 300 mg — ORALE — 2 times a day at 11:00 - 17:00 — effective: 12/04/2024, 11:00 — until further notice
- PARACETAMOLO — COMPRESSE** (Purple bar): DOSE: 500 mg — ORALE — if temperature is above 38.5 °C
- AMOXICILLINA — COMPRESSE — DISPERSIBILI** (Grey bar): DOSE: 10 mg — ORALE — once a day — effective: 13/04/2024, 08:00 — until 17/04/2024, 08:00
- OSSICODONE (OXICODONE)** (Grey bar): DOSE: 30 mg — once a day at 14:00 — effective: 12/04/2024, 14:00 — until further notice

Callouts highlight key features: 'Colour coded reminders' points to the status indicators; 'Current time' points to the vertical red line; and 'Conditional prescriptions' points to the Paracetamol entry.

# Body graph

## *Invasive medical device management*

- Body graphical indication for an easier management
- Different images according to sex
- Devices can be easily positioned on the body
- Activities can be performed for each devices positioned

The screenshot displays the DIGISTAT software interface for patient management. At the top, patient information is shown: TEST UNO, ALMA, 59 years old, with a date of the 24th of the month. The patient is in the 'RIANIMAZIONE' (Resuscitation) ward, room 3. The user is ADMIN Admin, and the time is 13:49 on 12 Jul 2024.

The central part of the interface features a body graph of a male patient. The graph shows various anatomical sites with colored markers: a blue dot on the chest and a yellow dot on the urethra. Surrounding the main graph are smaller views: a back view, a front view of the legs, and a 'NIV' (level) indicator.

On the right side, there are several filter buttons for different device categories: ACCESSI VASCOLARI, PRESIDI RESPIRATORI, SONDE GASTRICHE, PRESIDI URINARI, DRENAGGI, and STOMIE. Below these are buttons for 'IN SITU', 'RIMOSI', and 'CANCELLATI'. A 'Dispositivi' (Devices) table is also present, listing active devices with their IDs, names, locations, and types.

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An 'Attività' (Activities) section is also visible, showing a log of actions. A callout box highlights the table and activity sections with the text: 'Devices on the patient and related activities'.

# Printouts and reporting

*Finalizing process and using data collected to improve process*

# Daily Care Printout

Some examples from real installations

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\*Manual de seguridad de alarmas ©2014 ECRI Institute